## Manual Suction Pump

---



for quick suction of fluids

Intended to be left blank.

<b>ASSEMBLY</b>	..... 4
<b>DE HANDABSAUGPUMPE</b>	Gebrauchsanweisung ..... 5 - 6
<b>EN MANUAL SUCTION PUMP</b>	Instructions for use..... 6 - 7
<b>BG РЪЧЕН АСПИРАТОР</b>	Инструкции за употреба ..... 8 - 9
<b>CS RUČNÍ ODSÁVAČKA</b>	Návod k použití..... 10 - 11
<b>DA MANUEL SUGEPUMPE</b>	Bruganvisning ..... 11 - 12
<b>EL ΧΕΙΡΟΚΙΝΗΤΗ ΑΝΤΛΙΑ ΑΝΑΡΡΟΦΗΣΗΣ</b>	Οδηγίες χρήσης..... 13 - 14
<b>ES BOMBA DE SUCCIÓN MANUAL</b>	Instrucciones de uso ..... 15 - 16
<b>ET KÄSI-IMIPUMP</b>	Kasutamisejuhised..... 16 - 17
<b>FI KÄSIKÄYTTÖINEN IMUPUMPPU</b>	Käyttöohje ..... 18 - 19
<b>FR POMPE D'ASPIRATION MANUELLE</b>	Manuel d'utilisation ..... 19 - 20
<b>HR RUČNA USISNA PUMPA</b>	Upute za uporabu ..... 21 - 22
<b>HU KÉZI ELSZÍVÓPUMPA</b>	Használati utasítás ..... 22 - 23
<b>IT POMPA DI ASPIRAZIONE MANUALE</b>	Istruzioni per l'uso..... 24 - 25
<b>LT RANKINĖ ATSIURBIMO POMPA</b>	Naudojimo instrukcija..... 25 - 26
<b>LV MANUĀLĀS ATSŪKNĒŠANAS SŪKNIS</b>	Lietošanas instrukcija ..... 27 - 28
<b>NL HANDAFZUIGPOMP</b>	Gebruiksaanwijzing ..... 28 - 29
<b>NO MANUELL SUGEPUMPE</b>	Bruksanvisning..... 30 - 31
<b>PL POMPA RĘCZNA DO ODSYSANIA</b>	Instrukcja użycia..... 31 - 32
<b>PT BOMBA DE ASPIRAÇÃO MANUAL</b>	Instruções de utilização ..... 33 - 34
<b>RO POMPĂ MANUALĂ DE ASPIRAȚIE</b>	Instrucțiuni de utilizare ..... 34 - 35
<b>RU РУЧНОЙ АСПИРАТОР</b>	Инструкция по применению ..... 36 - 37
<b>SK RUČNÁ ODSÁVAČKA</b>	Návod na použitie ..... 38 - 39
<b>SL ROČNA SUKCIJSKA ČRPALKA</b>	Navodila za uporabo ..... 39 - 40
<b>SV HANDSUGPUMP</b>	Bruksanvisning..... 41 - 42
<b>TR MANUEL EL POMPASI</b>	Kullanım kılavuzu ..... 42 - 43
<b>SYMBOL DESCRIPTION</b>	..... 44 - 45

# ASSEMBLY

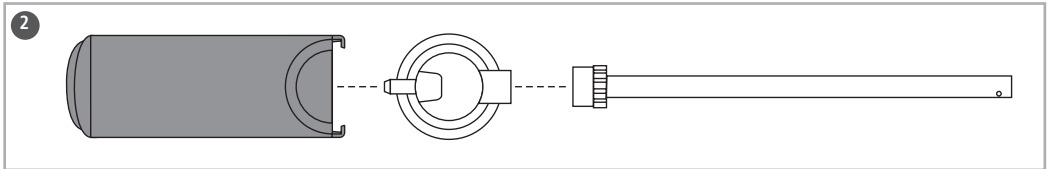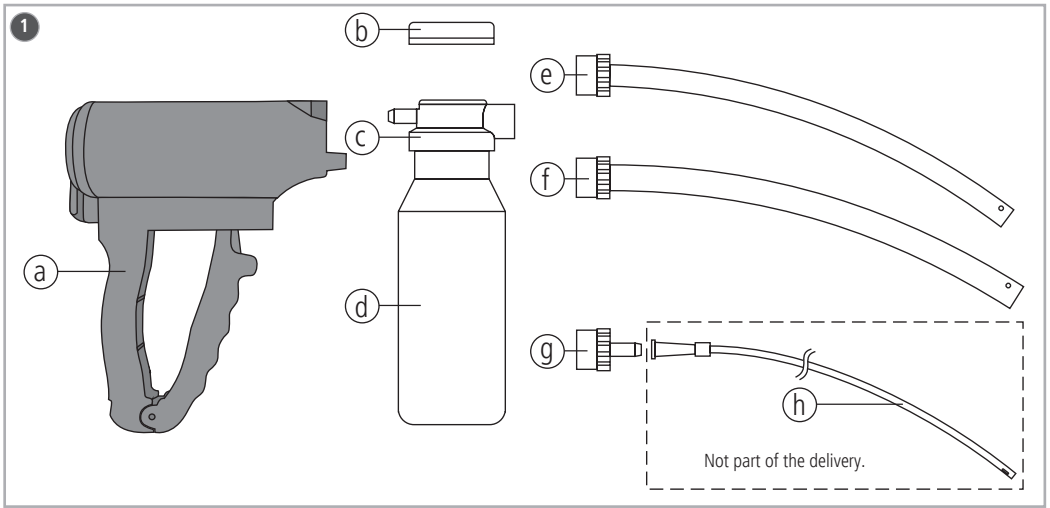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**English**

**INTENDED USE**

The Manual Suction Pump is used to provide a quick and effective suction of fluids from the oral cavity and the pharynx. The accessories are used for suctioning and collecting the fluids.

Clinical benefit: Reduces the risk to aspirate fluids from the oral cavity and the pharynx.

Patient target group: Patients who require suctioning of fluids from the oral cavity and the pharynx.

Place of use: hospital and pre-hospital conditions

**INDICATION**

- Aspiration of fluids from the oral cavity and the pharynx.

Further indications are not known.

**CONTRAINDICATION**

None known.

**SAFETY INSTRUCTIONS**

- Read and follow the instructions for use carefully before using the product.
- The product may only be used by medically trained personnel who have sufficient knowledge in handling the product.
- The user and/or patient must report all serious adverse events that occurred in connection with the product to the manufacturer and competent authorities of the EU member state (or report to the competent authorities of the respective country if an event occurs outside of the EU) in which the user and/or patient is located.
- The product is delivered ready to use. The vacuum pump must be cleaned thoroughly after every use (see "Wipe disinfection"). Damaged products must not be used.
- Maximum application time: 60 minutes
- No changes may be made to the product.
- Use original VBM accessories only. Using accessories from other manufacturers may have a negative impact on the function of the medical device. The manufacturer accepts no liability for this. Exception: Standard suction catheters are permitted.
- The product is not suitable for MRI.

- The product is not sterile.
- The Accessory Kit for Manual Suction Pump is intended for single use and must not be reused and/or reprocessed. Reprocessing influences the function of the Accessory Kit for Manual Suction Pump. Any reuse entails a potential infection hazard.

**PRODUCT DESCRIPTION (FIG. 1)**

a	Vacuum pump	Reusable
b	Sealing cap	Accessory Kit for Manual Suction Pump For single use
c	Suction cap	
d	Container 250 ml	
e	Suction catheter 28 Fr	
f	Suction catheter 40 Fr	
g	Adapter for standard suction catheter	
h	Standard suction catheter	

**PREPARATION PRIOR TO USE**

- ▶ Connect the container including the suction cap to the vacuum pump (Fig. 2).

## VISUAL AND FUNCTIONAL CHECK

- ▶ Check the product for damages and loose particles.
- Before using the device, check whether a vacuum is generated:
- ▶ Seal the opening of the suction cap with your thumb or palm of your hand.
  - ▶ Operate the vacuum pump handle several times.

This allows you to determine if a vacuum is generated.

The product must not be used if it did not pass the visual and functional check.

## USE

- ▶ Select the suction catheter according to the use and connect it to the suction cap (Fig. 2).
- ▶ Alternative: Connect the adapter for standard suction catheter to the suction cap. Then connect a standard suction catheter.

### CAUTION

- ▶ Carefully insert the suction catheter into the patient's oral cavity and pharynx.

To prevent fluids from contaminating or obstructing the vacuum pump, the container must be kept as upright as possible.

- ▶ Operate the vacuum pump handle.

### CAUTION

The container may be filled with max. 250 ml.

### CAUTION

If the vacuum pump can only be operated with difficulty, it should be checked whether there is an obstruction.

- ▶ Do not continue to suction until the obstruction has been eliminated.

- ▶ As soon as suctioning is complete, carefully remove the suction catheter from the patient's oral cavity or pharynx.
- ▶ Disconnect the filled container from the vacuum pump.
- ▶ Disconnect the suction catheter from the suction cap.
- ▶ Unscrew the suction cap from the container and cover with the sealing cap.
- ▶ If a lab test is required, the sticker on the container must be labelled.
- ▶ The used Accessory Kit for Manual Suction Pump must be disposed of completely (see "Disposal") and replaced with a new Accessory Kit for Manual Suction Pump REF 30-56-000.

## WIPE DISINFECTION

### CAUTION

- The vacuum pump must not be mechanically or manually reprocessed or sterilised.
- If wipe disinfection is not successful, or if the product is strongly contaminated, the vacuum pump must be disposed of as specified in section "Disposal".

Wipe disinfection must be performed with commercially available surface disinfectants based on alcohol, QACs (quaternary ammonium compounds) or hydrogen peroxide. When choosing a product for disinfection, a disinfectant with appropriate ranges of action must be used: bactericidal, yeasticidal and virucidal. After wipe disinfection, inspect the vacuum pump for visible contamination. If necessary, repeat wipe disinfection. After wipe disinfection, check the function of the vacuum pump (see "Visual and functional check").

## SHELF LIFE

### Vacuum pump

The shelf life of the product is 5 years.

### Accessory Kit for Manual Suction Pump

Use-by date: See product label

## STORAGE AND TRANSPORT CONDITIONS

### CAUTION

- Protect from heat and store in a dry place.
- Keep away from sunlight and light sources.
- Store and transport in the original packaging.

## SERVICE

Prior to the return of medical devices for complaints/repairs, the products must be cleaned to exclude any risk for the manufacturer's staff. For safety reasons, the manufacturer reserves the right to refuse soiled or contaminated products.

## DISPOSAL

The used or damaged products must be disposed of in accordance with the applicable national and international statutory regulations.

## PRODUCT SPECIFICATIONS

<b>Max. achievable vacuum</b>	480 mmHg
<b>Vacuum pump dimensions</b>	155 x 55 x 180 mm
<b>Weight REF 30-55-000 (incl. packaging)</b>	470 g

## SCOPE OF DELIVERY

REF	Description	Packaging unit
30-55-000	<ul style="list-style-type: none"><li>• Vacuum pump</li><li>• Accessory Kit for Manual Suction Pump</li></ul>	1 pc

## CONSUMABLES

REF	Description	Packaging unit
30-56-000	Accessory Kit for Manual Suction Pump	1 pc

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