

Intended Use

The Corius Seal™ wound dressing is intended to cover & protect acute wounds and absorb exudate.

User Information

The Corius Seal™ is a medical device and is indicated for use on all patients with acute wounds and who do not meet the contraindications described in this IFU. The expected clinical benefit of using the device is to cover and protect an acute wound, which helps to prevent further harm and lowers risk of infection, whilst absorbing exudate. For use up to 24 hours.

The Corius Seal™ is 20 x 15cm and is composed of adhesive hydrogel with a non-woven spunlace backing and removable plastic liners. The device has the following key features:

- Hydrogel adheres to skin but is easily removed.
- Removable plastic liners and non-woven spunlace backing aids with easy application and manipulation.
- Hydrogel absorbs wound exudate.
- Dressing is made from breathable materials.
- Robust performance under extreme environmental conditions.


Contra-indications

- Do not use on chronic wounds.
- Consult a clinical physician before applying to infected or heavily bleeding wounds, or if skin reaction occurs.


Warnings

- Excessively greasy skin will reduce the ability of the adhesive to function. If the patient's skin is suspected to be saturated with lipid-based lotion, clean with a skin cleansing wet wipe prior to application.
- If difficulty is experienced on removing the dressing, it should be irrigated or soaked with water or sterile saline solution.
- In case of intolerance, allergic reaction or irritation reaction to the dressing, remove and clean the area carefully, and seek medical advice.
- Dispose of the dressing appropriately or as per any local guidelines. Improper disposal risks cross-contamination.
- This product is intended for single use only. Reuse will risk cross-contamination and loss of product functionality.
- Using past shelf-life risks loss of functionality.
- Monitor the device and replace with a fresh dressing if it becomes saturated with fluid.
- Any serious incident that has occurred in relation to this device should be reported to the manufacturer (quality@safeguardmedical.com) and applicable competent authority.


Instructions for use

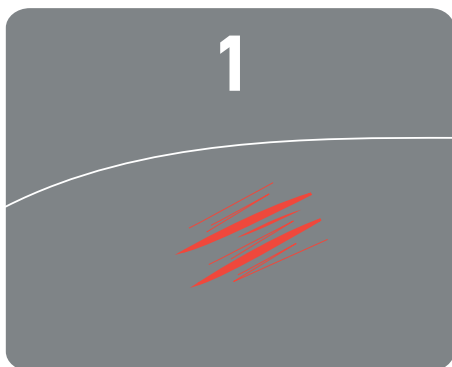

Before use, clean around acute wound. Note that excessively greasy skin will reduce the ability of the adhesive to function.

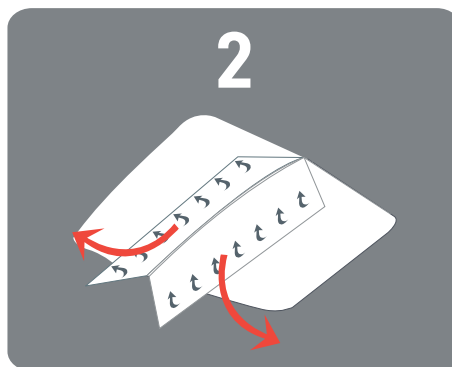

Remove the dressing from its packaging and unfold. Align the device so that it is directly over the wound and begin to peel back the plastic butterfly liners. If the wound cannot be covered by a single Corius Seal™ dressing, multiple dressings can be used by placing them edge to edge.

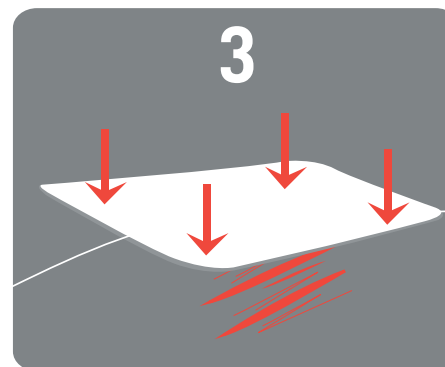

Whilst removing the butterfly liners, place the exposed adhesive surface onto the wound. Gently smooth down the dressing's adhesive area to ensure that the dressing is properly attached.



Made in USA



Non-sterile



Do not use if package is damaged



Single use only



Free of natural rubber latex



41°F / 5°C to 104°F / 40°C



Medical Device



Rev 1.0 - 11/20
DAN_0053_Rev0

Prometheus Medical Ltd,
Wicklow Serviced Offices,
Eden Gate Centre, Delgany Co.
Wicklow, Ireland



Prometheus Medical Technologies Ltd,
The Old Rectory, Hope Under Dinmore,
Herefordshire HR6 0PW, UK.

www.safeguardmedical.com



(01) 05060483570556