Medtronic

Microstream™ Advance


Adult-Pediatric Intubated CO₂ Filter Line

en	Instructions for Use	REF MVAIH
fr	Mode d'emploi	REF MVAIH100U
de	Gebrauchsanweisung	REF MVAIHH
nl	Gebruiksaanwijzing	REF MVAIHL
it	Istruzioni per l'uso	REF MVAI
es	Instrucciones de uso	REF MVAI100U
pt	Instruções de uso	REF MVAIHL
ru	Инструкция по эксплуатации	REF MVAI
da	Brugsanvisning	REF ZMVAI
sv	Bruksanvisning	REF ZMVAIH
el	Οδηγίες χρήσης	REF ZMVAIHH
no	Bruksanvisning	
hu	Használati útmutató	
pl	Instrukcja użycia	
cs	Návod k použití	
fi	Käyttöohjeet	
tr	Kullanma Talimatları	
sk	Pokyny na používanie	
sl	Navodila za uporabo	
ro	Instrucțiuni de utilizare	
bg	Инструкции за употреба	
et	Kasutusjuhised	
lt	Naudojimo instrukcijos	
lv	Lietošanas norādījumi	
hr	Upute za uporabu	
sr	Uputstva za upotrebu	

Table of Contents


English	10
French	11
German	11
Dutch	12
Italian	13
Spanish	14
Portuguese	14
Russian.....	15
Danish.....	16
Swedish	17
Greek.....	17
Norwegian	18
Hungarian	19
Polish.....	19
Czech.....	20
Finnish	21
Turkish	22
Slovak	22
Slovenian.....	23
Romanian.....	24
Bulgarian	24
Estonian.....	25
Lithuanian.....	26
Latvian.....	26
Croatian.....	27
Serbian.....	28

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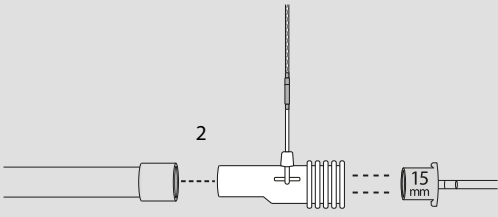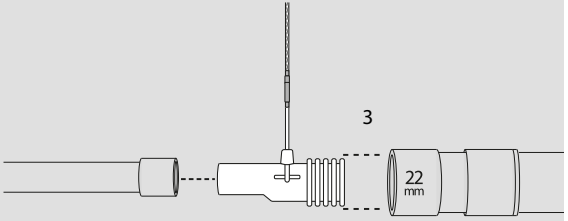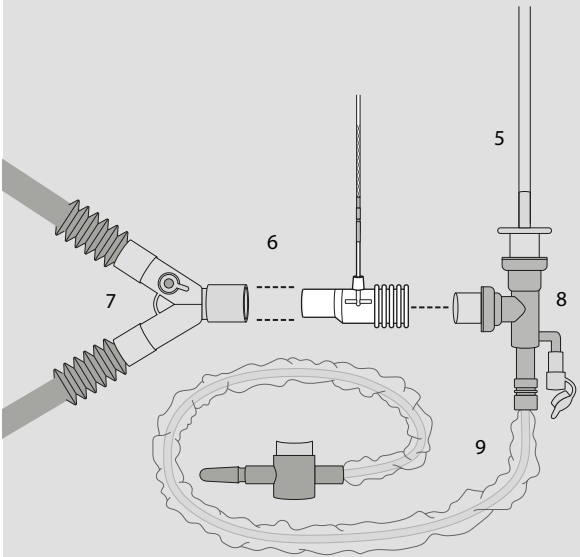
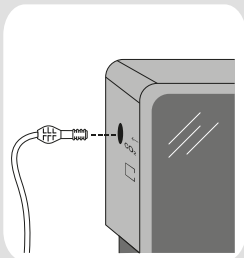
Microstream™ Advance

Adult-Pediatric Intubated CO₂ Filter Line

Indications for Use:

Intended to conduct a sample of the patient's breathing from a ventilator or anesthesia machine to a gas measurement device for measuring the percentage of CO₂ in the patient's exhaled breath. The set is intended for single patient use only. Intended population: Intubated Adult-Pediatric patients.

For use with all Microstream™-enabled monitors

Airway adapters for adult to pediatric patients:

- Added dead space <6.6 cc.
- To be used with an endotracheal tube > 4.5mm bore.
- Suitable for use with 22mm and 15mm patient tubing.

WARNINGS

- **Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components, turning the sampling line connector clockwise into the monitor CO₂ port until it can no longer be turned. Check connections for leaks according to standard clinical procedures, and ensure CO₂ values appear.**
- **Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line. Re-use of single-use accessories could pose a cross-contamination risk to the patient or damage the functioning of the monitor.**
- **When used with a suction catheter, do not place the airway adapter between the suction catheter and endotracheal tube. This is to ensure that the airway adapter does not interfere with the functioning of the suction catheter, and that the suction process does not damage the airway adapter. Damage of the airway adapter may harm the patient.**
- **Ensure no kinks are present in CO₂ tubing, as kinked tubing may cause inaccurate CO₂ sampling.**

Notes

- During setup, ensure that airway adapter can be easily attached and detached from the tubing before proceeding.
- During nebulization, lavage, or suction, follow monitor instructions for use (IFU) to suspend pump activity, in order to avoid moisture buildup and sampling line occlusion. In addition, especially if unable to suspend the CO₂ pump, disconnect the sampling line connector from the CO₂ port on the monitor.
- The response time of the 2m sampling line is ≤3.5 seconds, 4m sampling lines ≤5.9 seconds.
- Sampling lines with H in their names include a moisture reduction component (Nafion® or its equivalent) for use in higher humidity environments where long duration use of CO₂ sampling is required.
- Any serious incident related to device use that may occur should be reported immediately to the manufacturer, the local competent authority, and any other regulators as required.

TM Trademark of its respective owner.*

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Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic.
Other brands are trademarks of a Medtronic company.
© 2019 Medtronic. All rights reserved.
Oridion Medical 1987 Ltd., 7 Hamarpe St.,