emergency

Accuvac Rescue

Set 15792

Set 15830

*Montageanleitung
Assembly instructions
Notice de montage*

WEINMANN
medical technology

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
WM 66634b 06/11

These assembly instructions describe:

- fitting ferrite beads to the connecting cables for the power supply and to the battery of the Accuvac Rescue suction pump
- fitting a capacitor to the charging interface of the Accuvac Rescue suction pump

The ferrite beads and the capacitor are components of the Accuvac Rescue suction pump. Always also read the Accuvac Rescue service and repair instructions.

Installing the ferrite bead for the power supply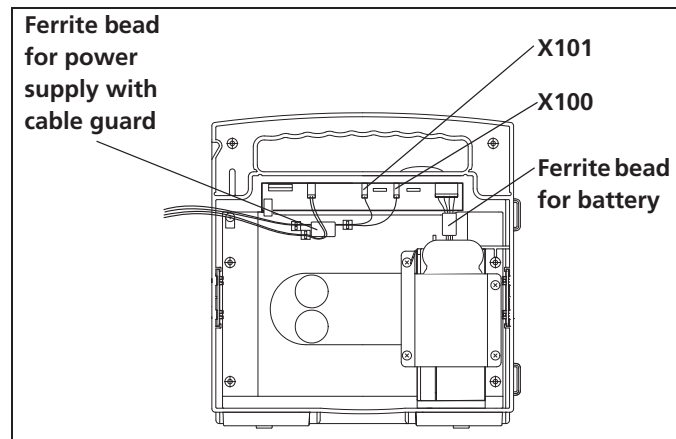


1. Place the ferrite bead for the power supply around the connecting cables for X101 and X100 and squeeze it together until it clicks.
2. Make sure the adhesive surface on the housing is free of dust, oils and grease.
3. Remove the adhesive film from the cable guard and glue the cable guard to the housing underneath the ferrite bead.
4. Remove the adhesive film from the cable holders.
5. Position the cable holders at the ends of the ferrite bead and glue them on.
6. Insert the connecting cables through the cable holders.

Installing the ferrite bead for the battery


1. Place the ferrite bead for the battery around the battery's connecting cables and squeeze it together until it clicks.

Fit capacitor for charging interface

1. Loosen and remove cover panel.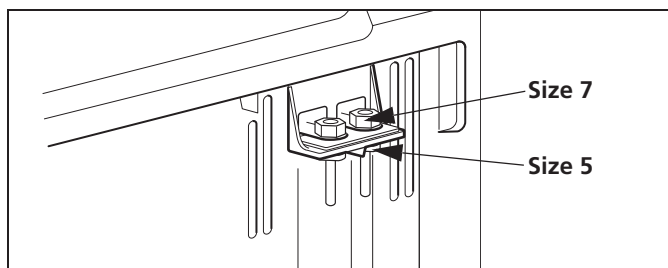


2. Use a size 5 wrench to stop the contact pins turning and undo the nuts using a size 7 ring wrench.
3. Pass the cable lugs for the capacitor to the outside. Use the same slots through which the cables to the charging interface pass.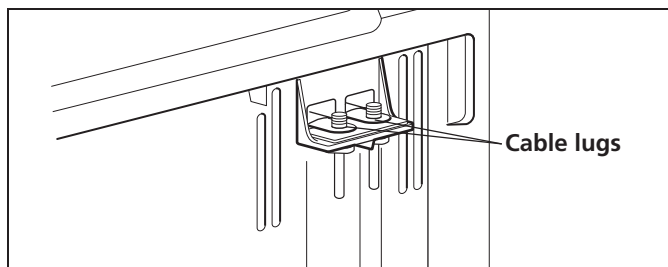

4. Put the cable lugs for the capacitor on the contact pins for the charging interface. In the process, please note: place cable lugs of identical color on top of one another (red on red, black on black).
5. Wet the threads of the contact pins with a drop of Loctite 222 (WM 14958 or WM 14915).
6. Place washers and nuts on the threads.

WM 66634b 06/11

7. Use a size 5 wrench to stop the contact pins turning and tighten the nuts using a size 7 ring wrench.
8. Clean the location for gluing on the mount with isopropyl alcohol.
9. Peel the protective film off the mount and glue the mount in position.

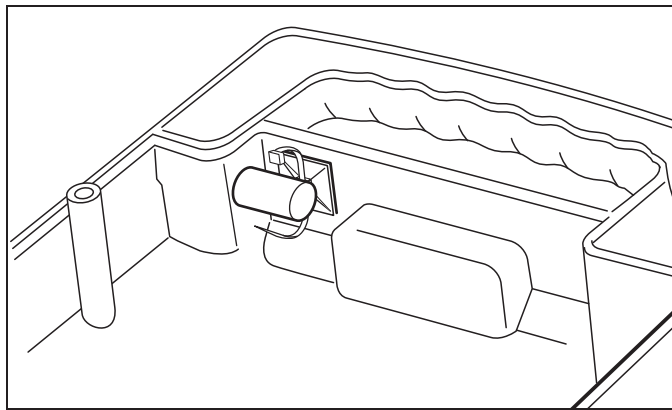10. Attach the capacitor to the mount using the wider cable tie.
11. Combine the capacitor cables with the cables for the charging interface. In the process, please note: use one cable tie to combine cables of the same color.

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Weinmann Geräte für Medizin GmbH+Co. KG